
Overview & Quick Reference Guide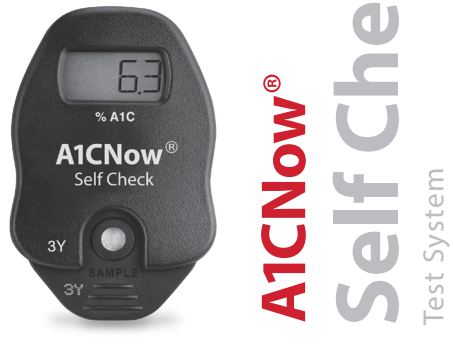

Note: All instances of the box or pouches in this insert are representative ONLY. Please refer to the packaging included with your specific kit.

1-877-870-5610 (toll-free) or +1-317-870-5610 (direct) We invite you to call us and we will guide you through the test.

IMPORTANT! FIRST, we strongly recommend you read all instructions carefully and watch the brief instructional video in our website at ptsdiagnostics.com. THEN, follow all steps as you perform the test.

Match Lot Numbers Use analyzer only with the materials included in the original kit. The analyzer will display "001L" if another test cartridge is inserted.

1 Open Plastic Shaker Pouch\* Do NOT open foil cartridge pouch until Step 6.

2 Stick Finger Wash hands with clean alcohol and dry before testing.

3 Collect Blood Dipper of cartridge in top of blood drop to fill. Make sure collector is adequately filled.

4 Insert Blood Collector Fully insert blood collector into shaker body. Push hard while twisting to fully insert.

5 Mix Shake vigorously 6-8 times (about 5 seconds). This will mix the blood with the solution.

6 Open Foil Cartridge Pouch\* AFTER Blood Collection Insert cartridge into analyzer immediately and use within 2 minutes.

7 Insert Cartridge "Click" test cartridge into place. Analyzer and test cartridge codes must match.

8 Prepare Shaker Remove shaker base. Wipe tip of shaker to be dry.

9 Dispense Sample into Cartridge Ensure analyzer is on level surface. Push down completely to dispense diluted sample and remove it in a continuous motion.

10 5 Minutes to Results Display counts down. QC read. Result. 4 tests left.

11 Dispose of Cartridge/ Save Analyzer Record your test progress here, and bring this with you when you see your doctor.

Result Interpretation Do not take any decision of medical relevance without first consulting your healthcare practitioner.

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